Made for Me™

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EN Contents

Stuff you need to know	1
Getting started	2
Dos and Don'ts	3
How to clean	3
Putting your pump together	4
Charge me up - powering your power unit	5
Using your breast pump	6
Storing breast milk	8
Remember	9
Health and Safety	9
Advice and FAQs	9
Device symbols	11

Mom, you're one of a kind.

Treat yourself like the true wonder you are and let us support you through your unique breastfeeding experience with the Made for Me™ range from Tommee Tippee.

Your Made for Me™ electric breast pump has soft, cushioned silicone cups that are gentle on your breasts. And it's powered by a lightweight, portable power pack so that you can use it wherever you need to.

Made especially for you because you're a one off...

Check out the rest of our Made for Me™ range, designed to help you on your breastfeeding journey.
www.tommeetippee.com.

Stuff you need to know...

IMPORTANT SAFEGUARDS

Please keep this instruction guide for future reference as it includes important information.

You can also find these instructions at tommeetippee.com.

- To protect against fire, electric shock and injury to persons, do not immerse motor unit, USB cable and adapter, or air tubes in water or other liquid.
- Do not place the power unit in a microwave oven or steam sterilizer.
- DO NOT WASH the air tubes or adaptor. This will result in bacteria build up and can lead to pump failure. Washing or sterilizing the air tubes will invalidate the warranty and will damage the power unit.
- Do not place the air tubes in a microwave or electric steam sterilizer or boiling water as they will melt.
- Misuse of the appliance may result in the risk of fire, electric shock and/or personal injury.
- Do not clean with solvents or harsh chemicals.
- This appliance should not be used or played with by children. Keep the pump, its air tubes and USB cable out of reach of children under 8 years old.
- The appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- User maintenance, other than cleaning and sterilizing is not required.
- Do not open the power unit, it contains no user serviceable parts and will

invalidate the warranty.

- Do not operate any appliance with a damaged USB cable or motor casing. If the appliance malfunctions or has been damaged please contact tommeetippee.com.
- If the damage is from new the unit must be returned to manufacturer.
- Do not leave the appliance in direct sunlight.
- Do not store below freezing or above 113°F.
- If damaged within a retail warranty period, this appliance should be exchanged at place of purchase. The manufacturer will then be made aware.
- Do not use any accessories other than those supplied with this appliance.
- The appliance should not be used adjacent or stacked with other equipment as the two appliances may interfere with each other.
- Do not allow appliance to come into contact with oil-based foods (e.g. oil/tomato based sauces) as staining will occur.
- This appliance is a personal care product and therefore cannot be returned once opened, unless faulty.
- Only use the adaptor and USB cable supplied with this product.
- Do not use while in transit.

For domestic use only - not intended for commercial or institutional use.

SAVE THESE INSTRUCTIONS

Any questions? Visit our website tommeetippee.com.

Getting Started

Mom, we know you have a lot to think about, but can we suggest that you get to know your breast pump before you need to start using it? It should make for a more relaxed experience, and you'll soon be an expert at this (as well as many other new things).

Get to know your breast pump

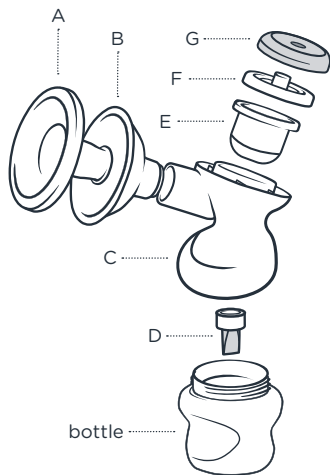
You may be wondering how both of your breast pump handsets fit together.

Don't worry, it will soon become second nature.

You can wash these parts:

- A. **Cup** - rounded, soft silicone part that fits into the horn to sit comfortably against your breast. Gently massaging to stimulate your milk flow.
- B. **Horn** - trumpet shaped part that supports the cup while you use the pump.
- C. **Body** - duck-shaped part that you hold and connects the horn and cup to your bottle.
- D. **Duck-bill valve (purple)** - small one-way valve that lets breast milk into your bottle, but doesn't let it back through as you move around.
- E. **Diaphragm** - small, transparent cup-shaped part that keeps the "wet" side and the "dry" side of the breast pump separate. This makes it a closed pump system.
- F. **Diaphragm cap** - seals to the diaphragm.
- G. **Lid (purple)** - allows you to connect the air tubes to the pump, and close the pump handset.

Together, these parts form your breast pump handset. You have two of these to form your double breast pump.

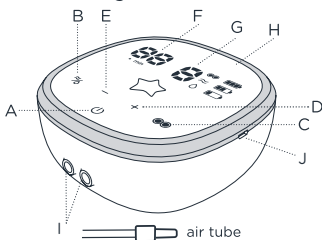
DO NOT wash these parts:

Air tubes – two plastic tubes with purple connectors at each end that attach your breast pump to the power unit.

Power unit – small, portable motor that powers the massage and expression features of your breast pump.

- A. **Power**  use to turn pump on and off, and pause during pumping.
- B. **Mode button**  switch between massage  and expression  modes.
- C. **Double/single pump**  switch between double and single pump modes.
- D. **Increase suction power** 
- E. **Decrease suction power** 

- F. **LED display** – shows time.
- G. **LED display** – shows mode level, there are 5 massage levels and 9 expression levels.
- H. **LED display** – shows battery charge level.
- I. **Air tube connector** – connect your air tubes to the power unit here.
- J. **USB socket** – connect this to a USB power source or adapter to recharge.

Dos and Don'ts

DO wash your hands with soap and water and dry thoroughly before handling your breast pump.

DO thoroughly clean and sterilize all the parts of your breast pump that come into contact with milk, following the instructions below. Do this before you use it for the first time and after every use.

How to clean

Separate each part of each pump handset and wash in clean soapy water. Don't use water that you've used for cleaning other products.

DO NOT wash the air tubes.

Rinse thoroughly with clean water.

You can also wash the pump handset parts in the top rack of a dishwasher. Place the purple duck-bill valve in the cutlery rack or dishwasher basket to keep it safe.

DON'T wash or sterilize the air tubes. This can cause bacteria to build up in the tubes and will damage the power unit.

DON'T wash, sterilize or heat the power unit. This is dangerous and will damage your power unit.

Sterilize all parts of both pump handsets using either cold water sterilization, microwave sterilization, boiling water, or electric steam sterilization.

DO NOT sterilize the air tubes.

For best results use Tommee Tippee electric or microwave sterilizers.

If you need to clean the power unit, USB cable or air tubes, wipe the outside with a clean damp cloth.

DO NOT wash any of these parts.

Putting your pump together

Each time you use your breast pump, check all parts of the pump and power unit for any damage.

Wash your hands with soap and water and dry thoroughly before handling your breast pump.

Make sure that all parts of the pump have been washed, cleaned and sterilized, and are completely dry.

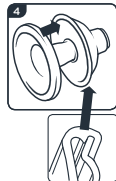
1. Place the clear diaphragm in the top of the body and push down gently to get a good seal.


2. Place the clear diaphragm cap over the diaphragm and push down gently to seal.

3. Place the purple lid over the top of the body and turn it clockwise until it clicks closed.

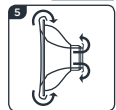4. Push the soft, silicone cup into the widest part of the horn and smooth its lip evenly over the edge. Work your way all around the rim to make sure the cup fits smoothly, evenly and securely. The two dimples in the cup should be clear to allow air into the horn.

5. Now smooth the lip of the silicone cup around the narrow end of the horn in the same way.

Once you've fitted and used it, you don't have to take the silicone cup out of the horn unless milk or dirt becomes trapped inside. Then you can ease the edges of the cup from the horn and gently pull them apart for cleaning.

6. Push the horn with cup attached onto the body until it's fully engaged and secure. **Turn your pump handset upside down.**

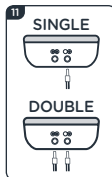
7. Insert the purple duck-bill valve. Gently squeeze the sides of the valve to make sure it's open.

8. Screw in a Tommee Tippee Closer to Nature or Advanced Anti-Colic bottle. This breast pump is designed to fit all Tommee Tippee Closer to Nature and Advanced Anti-Colic bottles with adaptor rings (excludes Tommee Tippee Closer to Nature glass bottles).

9. Repeat for the other breast pump handset.

10. Place the assembled pumps on a flat surface. Connect one end of the air tube to the purple lid of each pump body and the other to the power unit. The air tubes go into the two holes at the front of the unit.


11. Insert the other end of the air tubes into both sockets on the front of the power unit. If you want to use the pump in single mode only insert the air tube into the single socket marked .

Charge me up - powering your power unit

The power unit for your breast pump uses a battery similar to the one in your mobile phone. Charge it up so that you can power your breast pump anywhere. You do not need to have your power unit plugged in to mains power to use your breast pump. The power unit will come with some charge in it, but you should fully charge it before first use.

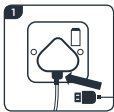To charge the power unit

1. Connect one end of the USB cable into your power unit and the other end into the supplied adapter.

2. The LED battery icon will flash to show that your power unit is charging. Charge until all 3 bars are full. Charging time may vary depending on voltage and equipment used, from 3 - 6 hours.

3. When it's fully charged all three of the LED lights will remain on.

What do the symbols mean?

Battery fully charged.

Battery Charging.

Low charge - you have up to 15 mins of power left.

Massage mode - a gentle massage rhythm that mimics the way your baby latches to your nipple to help you begin to produce milk.

Express mode - regular pumping action to express milk.

Double mode - double circle is visible to show pump is operating in double mode. This symbol is not visible in single mode.

Level Massage - 5 levels.
Express - 9 levels.

Timer - allows you to monitor how long you have been pumping for. In pause mode the timer will stay lit but stop counting. The power unit will automatically switch off after 30 minutes.

Using your breast pump

You've cleaned and assembled your breast pump, powered up your power unit. You're prepped and ready to go...

The best time to use your breast pump is after the first feed of the day (unless your breastfeeding advisor has told you something different).

1. Hold a pump handset around the narrow neck of the body. Hold it in whichever hand feels most comfortable. As you use it, you may find it's more comfortable to cup your hand around the bottle. Do not hold it by the air tube.

2. Place the soft silicone cup attached to the horn over your breast so that it creates a seal all the way around with your nipple at the center of the hole, you don't need to cover the whole of your areola or breast with the horn, so don't force it. It should sit comfortably. If it doesn't feel right, relax and reposition.

3. Once you're ready, press and hold the power button for a couple of seconds. The pump will start with a gentle massage rhythm that mimics the way your baby latches on to your nipple to help you begin to produce milk. It will give you 10 short pulses and one longer pump.

4. Choose whether you want to use the pump in single or double mode. Two circles on the LED display show that you've selected double mode. If you're using the double pump mode, place the second pump handset over your other breast as described above, or into your pumping bra.

Make sure hands are washed and dried before touching your breast pump, and breast pump parts are washed and sterilized.

Get yourself in a comfortable position, sitting, standing or half-lying. You may choose to wear a pumping bra (available separately) to help keep your hands free. The important thing is to keep the pump handsets vertical while you hold them against your breasts. You'll also need a flat surface nearby to put the pumps and bottles on when you're done.

5. After about a minute, the pump will automatically move to more powerful massage settings. If you want to move on onto a more powerful setting, touch the "+" button.

6. Once you feel or see your milk, you can switch to Express mode. This gives you a more regular and prolonged pumping action to help you express milk. Press the mode selection button once to switch to Express mode.

7. To move to a more powerful and longer express setting, touch the "+" button. To move to a less powerful and shorter express setting, touch the "-" button. The power will automatically shut off after 30 minutes of express mode.

8. If you want to pause at any point, briefly touch the power button. This will pause the timer as well. Briefly touch again to resume pumping.

When you want to stop expressing

- When your milk is no longer freely flowing, touch and hold the power button for a couple of seconds to turn off the pump.

- Disconnect the air tubes from the purple lids and gently unscrew your bottle from the pump.
- You can store your breast milk in this bottle, by adding a clean sterilized lid or nipple and hood (see Page 8 for storage advice). Or decant it into another clean, sterilized bottle or pouch to feed it to your baby.
- When you've finished using the pump, disassemble all the parts and clean and sterilize as outlined in the cleaning section on Page 3. Don't clean or sterilize the air tubes.
- Plug your power unit into the adapter provided or a USB charging point to recharge it ready for next time.
- If you want to pause during pumping to change position, or move down to a lower setting that's absolutely fine. It's okay to remove the pump from your breast at any time. You can always come back to it later.
- You can use the built-in timer to monitor how long you're expressing for. The power unit will automatically shut off after 30 minutes of expressing.

If using this breast pump is more painful than when you breast feed, stop using it immediately.

When pumping, your nipple should sit comfortably in the center of the silicone cup and will be extended down the tube by gentle suction. The pump creates a seal around your areola and nipple. You do not need to cover the whole of your areola or breast with the horn. Do not force your breast or nipple into the horn.

Don't tilt the bottle while expressing. Keep it upright to keep your milk flowing into the bottle.

You don't need strong suction to express milk from just behind your nipple. Use the mode and power setting that feels most comfortable for you. This may be different at different times of day or at different stages of your breastfeeding experience.

The illuminated LEDs and quiet pump unit allow you to use the pump at night without disturbing others when you want to keep the lights low.

Storing breast milk

Method of storing milk	Use within	REMEMBER
Room temperature <77°F	6 hours	<ul style="list-style-type: none">• Never defrost or heat breast milk in a microwave• Never refreeze breast milk• Once defrosted, the breast milk should be kept in the fridge and used within 12 hours• Do not boil breast milk• Throw away any leftover breast milk after feeding
Refrigerated at 40-50°F	3 days	
Refrigerated at 32-39°F	6 days	
Refrigerated before freezing	24 hours	
Once frozen	6 months	
Defrosted before feeding	12 hours	

- When you want to freeze breast milk, allow space at the top of your bottle for it to expand. It's easiest to freeze breast milk in individual feed quantities of 2-4oz / 60-125ml.
- Your breast milk may separate out into layers when you store it. Give it a shake to remix it thoroughly before feeding your baby.
- Thaw frozen breast milk in the fridge or by standing your bottle in lukewarm water until it has reached body temperature. Or use a Tommee Tippee bottle warmer.
- If you want to take expressed milk out and about with you, make sure it stays cold or frozen until you need to feed your baby.

Adapter:

Input: 100-240~, 50/60 Hz, 0.4 A Max

Output: 5.0 V, 2000 mA

Main unit: 5 V, 1.5 A)

Remember

DON'T

- ✗ Don't use while driving or operating machinery.
- ✗ Don't use while bathing or showering – it is not waterproof.
- ✗ Don't use whilst sleeping.
- ✗ Don't use in temperatures below 50°F.
- ✗ Don't use if the USB cable or motor unit is damaged.
- ✗ Don't share your breast pump with other moms.
- ✗ Don't leave the USB plugged in when the unit is not charging or in use.

Health and Safety

- Don't place furniture or other obstructions on the USB cable or air tubes, and make sure no one can step on or trip over them. Keep them well out of reach of your baby and any young children.
- Keep the USB cable and air tubes away from hot surfaces such as radiators and central heating pipes.

Advice and FAQs

Get comfortable with your breast pump

- Unless you've been told otherwise, the best time to try your breast pump for the first time is after your baby's first feed of the day.
- Make sure you're in a comfortable position and try to relax.
- You're not trying to suck milk out. It comes from just behind your nipple. Start with the mode and power that feels most comfortable and allows you to express your milk.
- Using your breast pump shouldn't hurt. If it does, try a gentler setting. Then check your breasts. If you have any red or hot areas, stop using the breast pump and get in touch with your breastfeeding advisor as soon as possible.
- Every mom is different. You may only produce a little milk, or flow freely. You may produce more milk at different times of the day (morning or evening) or different volumes from one breast to another. The Tommee Tippee electric breast pump allows you to choose different settings to suit you at different times.
- Don't continue using the pump once your milk has stopped coming. Give your breasts a break.

FAQs

Help – there's no or very little suction. What can I do?

It's most likely that there's some air getting in somewhere, which means the pump won't work properly.

- Check that each air tube is connected securely at both ends and there are no kinks in it.
- Make sure that the soft silicone cup is securely attached to the horn at the top and bottom and forms a comfortable seal around your breast. Adjust your position if you need to.
- Check each duck-bill valve is clean and opens easily. Give it a gentle squeeze if it's stuck.
- Check each diaphragm and diaphragm cap are secure and there are no signs of damage.

Will milk get sucked into the power unit?

No. If you have fitted the diaphragm and cap properly, the air tube and pump are separated from your milk and can't come into contact with the power unit. It's what's known as a closed pump.

Help, the pump is making a funny sound

Check if you have selected single or double mode. And that you have the air tube/s connected correctly. If you are using a single pump and have the double mode selected (shown by the double circle lit up on the LED display), or have a single air tube attached to the wrong connector, the pump will run continuously, and make a noise when it's trying to provide suction to the unattached pump handset. Choose the correct mode and connection for single or double pumping.

Why do I have to use a Tommee Tippee bottle?

The Tommee Tippee Made for Me™ electric breast pump is designed to be a perfect fit for Tommee Tippee bottles and pouches (excludes Tommee Tippee Closer to Nature glass bottles). You may get leaks and spills if you use other brands of bottles and containers that don't fit correctly.

WARNING

No OVER-CURRENT RELEASES or FUSE is needed since only consumer grade batteries are used, which have limited power output.

Warning against servicing and maintenance while the ME EQUIPMENT is in use.

Disposal of waste products, residues, etc., and of ME EQUIPMENT and ACCESSORIES at the end of their EXPECTED SERVICE LIFE are identified in the instruction for use.

Warnings identifying nature of HAZARD when replacement of a component could result in unacceptable RISK, and when replaceable by SERVICE PERSONNEL all information necessary to safely replace the component

No OVER-CURRENT RELEASES or FUSE is needed since only consumer grade batteries used, which has limited power output.

Device Symbols

IP21

IP21 Indicates that the unit is resistant to condensation and finger entry.

Compliance with Federal Communications Commission Requirements.

Keep the device dry.

The product contains a Lithium Ion Battery.

Voltage 5v=
Power 10W

The power and voltage consumed by the device when plugged into a USB socket.

The product is double insulated (Class II Appliance).

In some countries this is a medical appliance and this indicates that this product has Applied Parts.

Warning!

Serial number.

Read the manual carefully

1. This equipment has been tested and found to comply with the limits for medical devices in EN60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.
2. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation.
3. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
 - Reorient or relocate the receiving device.
 - Increase the separation between the equipment.
 - Connect the equipment into an outlet on a circuit different from that to which the other device(s) are connected.
 - Consult the manufacturer or field service technician for help.

Transport and Storage Condition

Temperature: 32°F to 113°F

Humidity: 10% - 85%

Atmospheric pressure: 850Hpa - 1060Hpa

Working condition

Rated temperature range: 50°F to 104°F

Rated relative humidity: 10% - 85%

Rated atmospheric Pressure: 850Hpa - 1060Hpa

Long-term storage should be kept in a dry and cool place.

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Guidance and Manufacturer's Declaration

Important information regarding Electro Magnetic Compatibility (EMC)

This electrical medical equipment needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual; The equipment conforms to this IEC 60601-1-2:2014 standard for both immunity and emissions. Nevertheless, special precautions need to be observed:

- The equipment with no ESSENTIAL PERFORMANCE is intended used in Home healthcare environment
- **WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally”.
- The use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the 1183, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.”
- **WARNING:** If the use location is near (e.g. less than 1.5 km from) AM, FM or TV broadcast antennas, before using this equipment, it should be observed to verify that it is operating normally to assure that the equipment remains safe with regard to electromagnetic disturbances throughout the expected service life.

EMI Compliance Table (Table 1)

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	Home healthcare environment
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	Home healthcare environment

EMI Compliance Table (Table 2-5)

Table 2 - Enclosure Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3 30A/m
Rated power frequency magnetic fields	IEC 61000-4-8	50Hz or 60Hz

Table 3 – Proximity fields from RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Home healthcare environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ± 5 kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810		
870	800-960	Pulse modulation 18Hz, 28V/m
930	1700-1990	Pulse modulation 217Hz, 28V/m
1720		
1845		
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240	5100-5800	Pulse modulation 217Hz, 28V/m
5500		
5785		

Table 4 – Input a.c. power Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrical fast transients burst	IEC 61000-4-4	± 2 kV 100kHz repetition frequency
Surges Line-to-line	IEC 61000-4-4	± 0.5 kV, ± 1 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands and amateur radio bands between 0.15MHz and 80MHz 80MHz at 1kHz
Voltage dips	IEC 61000-4-11	0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% UT: 1 cycle and 70% UT; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% UT; 250/300 cycles

Table 5 – Signal input/output parts Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home healthcare environment
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands and amateur radio bands between 0.15MHz and 80MHz 80%AM at 1kHz

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Let us know what you think

We'd love to hear what you think of the Made for Me™ range. Share your photos, reviews and tell us about your experiences on our social channels.

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★ **tommee tippee**
helping you ParentOn™

tommeetippee.com

Distributed by; Distribuido por;
Mayborn USA Inc.
1010 Washington Boulevard, 11th Floor,
Stamford, CT 06901

TOMMEE TIPPEE is a registered trademark.
Copyright© Mayborn USA Inc., 2020.
Mayborn USA Inc., retains all intellectual
property rights to this product.

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Copyright © Mayborn USA Inc., 2020.
Mayborn USA Inc., posee todos los derechos