




ultra uv 

**3 in 1 UV** sterilizer,  
dryer and storage

### INSTRUCTIONS FOR USE

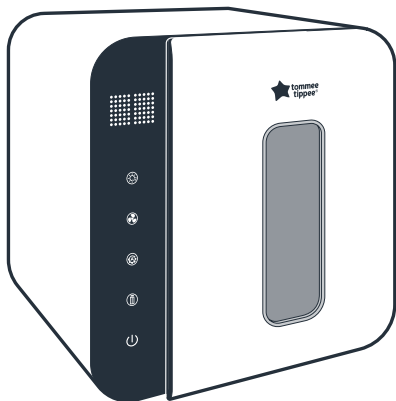

Please read and keep these instructions

522284  
ZTD10XB-8016

# IMPORTANT SAFEGUARDS READ ALL INSTRUCTIONS FOR HOUSEHOLD USE ONLY

These instructions can also be found at [www.tommeetippee.com](http://www.tommeetippee.com)  
Keep this instruction sheet for future reference as it includes important information.

## WARNING

To reduce the risk of fire, electric shock or injury. Do not leave equipment when plugged in. Unplug from outlet when not in use and before servicing. Do not pull or carry by cord, use cord as a handle, close a door on cord, or pull cord around sharp edges or corners. Keep cord away from heated surfaces. Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord. Do not handle plug or equipment with wet hands. Turn off all controls before unplugging. Do not allow to be used as a toy. Keep the appliance and its cord out of reach of children. Children shall not play with the appliance. Do not immerse cord, plugs or other parts of the unit in water or other liquid. Misuse of the roduct may result in the risk of fire, electric shock and/or personal injury. This appliance is intended for household use only. Do not use outdoors. Before plugging in the sterilizer for the first time ensure that the voltage is compatible with your electricity supply by checking the label on the base of the unit. Appliances can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. User maintenance other than cleaning and descaling is not required for this product. Cleaning and user maintenance shall not be made by children. The surfaces are liable to get hot during use.

Always use on a flat heat resistant surface. Do not move appliance when in use. Use in a well ventilated area.

Do not sterilize electrical components. Do not touch hot surfaces, allow to cool before handling. Do not operate any appliance with a damaged cord or plug or after the appliance malfunctions or has been damaged in any manner. If the supply cord is damaged it must be changed by a qualified person in order to avoid a hazard. Do not let the cord hang over edge of kitchen work surface or touch hot surfaces. Do not use any accessories other than those supplied with this appliance. This appliance contains a UV emitter. Do not stare at the light source. If the UV light remains on when the door is open or if the door is damaged or missing, please do not use the sterilizer. Follow the instructions when changing the UV light.

### **Information on the disposal for Waste Electrical & Electronic Equipment (WEEE)**

Used electrical and electronic products should not be mixed with general household waste. For proper disposal for treatment, recovery and recycling, please take these products to designated collection points.

In some countries you may be able to return your products to your local retailer upon the purchase of a new product. Disposing of this product correctly will help you save valuable resources and prevent any possible effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for details of your local collection point.

## **SAVE THESE INSTRUCTIONS**

## USAGE

Always use the sterilizer on a flat heat resistant surface and keep it away from inflammables and explosives.


Place it at least 4 inches away from a wall.

Do not let the cord hang over the edge of kitchen work surface or touch hot surfaces.

Do not move the sterilizer when in use.

Do not cover the sterilizer and keep your hands, face and other body parts away whilst using.

## PARTS GUIDE

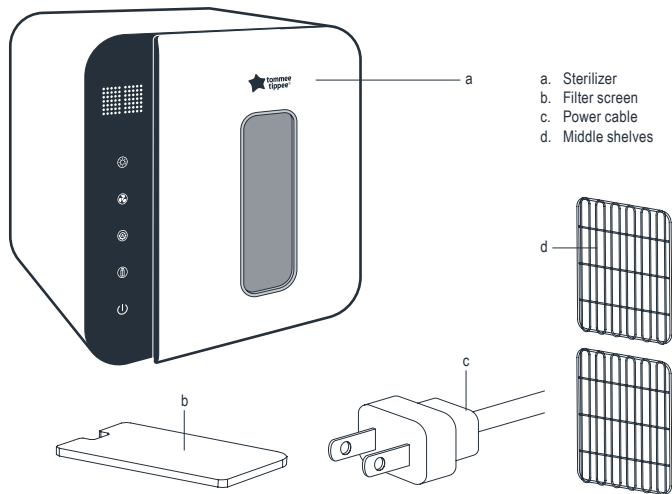

### Setting up your sterilizer before use.



**Clean your sterilizer before first use. Refer to parts guide and diagram.**

**Do not connect to power at this stage.**

1. Remove all parts from the box
2. Place the shelves inside
3. Wipe clean your sterilizer before use with a damp cloth
4. Plug in the power cable

## HOW TO USE

- Wash items you want to sterilize in warm soapy water, rinse and drain before placing in the sterilizer.
- Don't put dripping wet items into the sterilizer, as water can gather at the base. If there is water at the base or leaking from the sterilizer, please turn off the machine and wipe away water.
- Place the items inside the sterilizer. Separate all components and ensure all surfaces that need to be sterilized are visible.

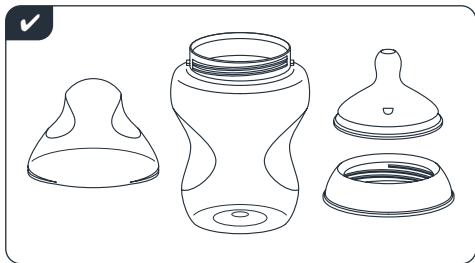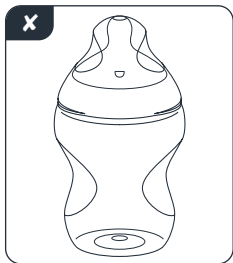

- Place bottles facing upwards to help with the drying function.
- Plug into the power, press the on/off button then select the function you wish to use – sterilize only, dry only, auto sterilize and dry or storage function.
- The screen will illuminate and flash to show the function when in use. The countdown timer will show how long is left in the cycle.
- Do not take items out of the sterilizer during sterilization.
- When in use, hot air is released from the sterilizer. Please keep your hands, face and other body parts away.
- The sterilizer uses UV light which can cause damage to your eyes and skin. Please don't stare at the light source.
- When the cycle is finished, the countdown will show 0 and emit a sound.
- Remove your items from the sterilizer. Be careful as they may still be warm, especially after drying.
- If you wish to leave the products inside the sterilizer you can use the storage mode which will keep them sterile.
- The UV lamp bulb will be hot after sterilization. Do not touch.
- There may be some residual heat after use. Do not touch hot surfaces, such as the shelf, until these have cooled down.
- You may experience a faint odour on first use. This is normal.

Baby bottles that have been sterilized in the UV sterilizer may become slightly yellowed in appearance. This is nothing to worry about. The bottles are still safe to use.

## TROUBLESHOOTING

- Remember - if you open the door to the sterilizer at any point during the cycle then the cycle will pause. If this happens during sterilization its best to start that cycle again.
- To stop a function at any point press the button that is in use until the lights are no longer illuminated.
- If the sterilizer doesn't work, allow it to cool for 5 minutes then try to restart.
- If any items block the outlet unplug the sterilizer, allow it to cool for 5 minutes, then remove all items and try again.

## FAQs

### Why do I have to position my sterilizer away from the wall?

We recommend a 4 inches gap to allow for ventilation, this protects your work surfaces and the unit.

### If I leave my bottles in storage mode, how long will they remain sterile for?

In storage mode the sterilizer will sterilize and dry the product every 2 hours, until the sterilizer is switched off with the power off button.

### How often should I rinse the sponge filter?

This very much depends on your environment. We would recommend that you wash or replace your sponge filter when you start to notice any dust build up on the sponge, make sure you check the sponge filter regularly.

### I've lost my sponge filter / damaged it – where can I get a replacement?

You get a spare sponge filter with the product, you can use this spare if you have lost or damaged the original sponge filter.

### How long will the UV Light Bulb last?

Your Philips G4 TUV 4W T5 bulb will last 6,000 hours before it needs replacing.

## FUNCTIONS



### Sterilize only function

Press this button once for 35 minutes sterilization and twice for 45 minutes sterilization.



### Drying only function

Press this button once for 30 minutes drying, twice for 40 minutes drying and three times for 50 minutes.



### Auto mode

Press this button to start a drying then sterilizing cycle for 60 minutes in total, drying only first for 25 minutes then sterilizing and drying for 35 minutes.



### Storage function

Press this button alongside any of the functions above to allow items to be stored in the sterilizer. The sterilizer will run a programme that emits 5 minutes of UV light and 5 minutes of drying every 2 hours, to ensure items inside remain sterile. You can open the door and remove one product, then press the storage button again to keep the remaining items inside sterile.



### Power on/off

Press this button to switch the sterilizer on and off.

An alert sounds when you press any of the buttons and when you open or close the door. You can turn off the sound on the buttons but not when you open or close the door.

#### Button sound ON:



#### Button sound OFF:



## CLEANING & MAINTENANCE

You do not need to carry out any maintenance to this product other than cleaning. Descaling is not necessary. Do not allow children to clean or maintain this product.


Make sure the sterilizer is cool before cleaning.

Change the filter regularly, you can replace using the spare filter sponge provided and reusing filters by rinsing with warm water and allowing to air dry.

We recommend that you clean your sterilizer regularly, every 1-2 days to maintain its performance.

1. Please turn off the power supply before cleaning and wait for your sterilizer to cool down.
2. Use a clean damp cloth to clean the inner and outer walls of the sterilizer. Do not put the sterilizer into water. Do not use vinegar or other chemicals to clean as these can cause damage.
3. Wipe away any dust on the screen with a clean damp cloth.

## HOW TO REPLACE THE FILTER SCREEN

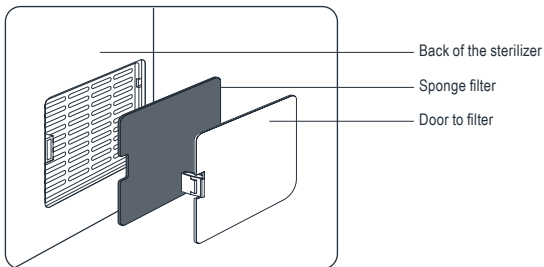


1. Open the filter screen at the back of the unit
2. Remove the old sponge filter
3. If you wish to re-use the sponge filters, then rinse the sponge in warm water and leave to air dry.
4. Alternatively take your new sponge filter out of the packaging or a sponge filter you have cleaned and place into the space. There is only one way the sponge can fit into the gap.

## CHANGING THE UV LIGHT BULB

Turn off the power and unplug the sterilizer.

Ensure the bulb is cool to touch.

Ensure you are using a Philips G4 TUV 4W T5 Mini.

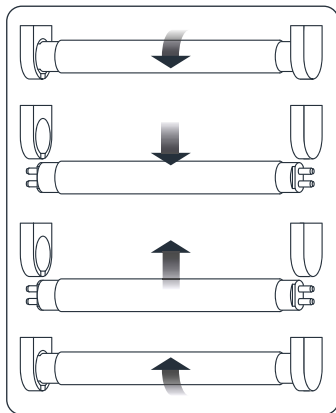

1. Rotate the bulb 90 degrees anti clockwise
2. The socket should now be vertical, and you can remove the bulb by pulling it downwards
3. Take the new bulb and push it up into the vertical slots of the socket
4. Rotate the bulb 90 degrees clockwise
5. The socket will now be in the locked horizontal position

## TIPS

1. UV bulb contains the watermark which may be hazardous to the environment.  
Do not crush, break, or open the tube.  
Do not leave the UV bulb with the household garbage stacked together.
2. If you have any recycling questions, please contact your local waste management office.

## ELECTROMAGNETIC FIELDS

This product complies with all relevant exposure to magnetic fields of the applicable standards regulations.



Mayborn USA Inc.  
1010 Washington Boulevard,  
11th Floor, Stamford,  
CT 06901

Mayborn USA Inc., retains all intellectual property rights to this product.

TOMMEE TIPPEE®. Copyright © 2021.

Made in China.

Model number: ZTD10XB-8016.  
Voltage: 110V 60Hz 67W



Any questions?  
Visit our website  
[tommeetippee.com](https://tommeetippee.com)

