US

Before each use, hold tip of valve (C) up to the light to see the slit and pinch apart to ensure proper venting.

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US

### closer to nature® anti-colic bottle & nipple

Before using your closer to nature bottles or nipples, please ensure that you read these instructions thoroughly.

Please keep this instruction sheet for future reference as it includes important information.

#### FOR YOUR CHILD'S SAFETY AND HEALTH

##### **WARNING!**

Always use this product with adult supervision. Do not leave baby alone with drinking equipment due to the risk of choking, baby falling or if the product has disassembled.

Never use feeding nipples as a pacifier.

Continuous and prolonged sucking of fluids will cause tooth decay. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle for long periods through the day and particularly through the night, when saliva flow is reduced or if it is used as a pacifier.

Always check food temperature before feeding. Do not heat in a microwave.

Always stir heated food to ensure even heat distribution and test the temperature before serving.

For your child's safety – Always check food temperature before feeding by testing on a sensitive part of your skin. For example on the inside of your wrist.

#### CLEANING

Before first use, clean the product.

Clean before each use.

Always wash the product immediately after use.

Wash in hot soapy water and rinse in clean water or use a sterilizer after washing.

Do not clean with solvents or harsh chemicals.

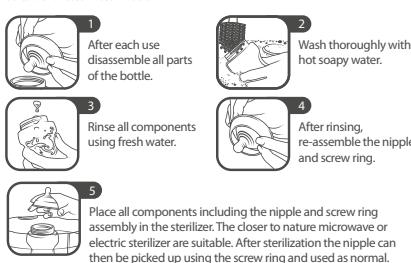
Top rack dishwasher safe.

Do not allow product to come into contact with oil based foods (e.g. oil/tomato based sauces) as staining will occur.

Do not turn the nipple inside out for cleaning. Damage could result. Do not use a bottle brush. Only the nipple and spout cleaner of the closer to nature Bottle & Nipple brush is suitable.

#### STERILIZING

Before first use place in boiling water for 5 minutes. This is to ensure hygiene. Suitable for steam sterilization.

#### ASSEMBLY OF YOUR ANTI-COLIC BOTTLE

1 Insert the heat sensing vent tube (A) into the vent wheel (B)

2 Fit valve (C) to end of heat sensing tube (A). If the valves are sliding off, place them in freshly boiled water for 5 minutes.

3 Do not push on too far as damage could result.

4 Fit nipple (D) into the collar (E) and screw onto the bottle.

a Due to the larger size of Tommee Tippee's new nipple and screw ring, care needs to be taken during assembly to ensure that the nipple is in the correct position shown (diagram a).

b The best method of assembly of the nipple and screw ring is to carefully pull the nipple through the ring in different directions to ensure the silicone lip fully clips' over the edge of the screw ring. The area where the nipple and screw ring meet should be flushed as shown (diagram b).

WHEN MAKING UP FEEDS DO NOT USE BOILING WATER. BOILING WATER SHOULD BE ALLOWED TO COOL FOR 45 MINUTES BEFORE PLACING THE HEAT SENSING TUBE INTO THE BOTTLE

#### STORAGE AND USE

Inspect before each use and pull the feeding nipple in all directions. Throw away at the first signs of damage or weakness. Do not leave a feeding nipple in direct sunlight or heat, or leave in disinfectant ("sterilizing solution") for longer than recommended, as this may weaken the nipple. Do not store with or allow to come into contact with solvents or harsh chemicals. Not suitable for use in a grill oven or a microwave oven with the grill feature switched on. Keep all components not in use out of reach of children. For safety and hygiene reasons it is recommended the feeding nipple is replaced at least every 2 months. Do not heat in a microwave.

Distributed by Mayborn USA Inc.

1 Edgewater Drive

Suite 109

Norwood, MA 02062

Copyright© Mayborn USA Inc. 2010-13

TOMMEE TIPPEE and CLOSER TO NATURE are registered trademarks.

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