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Distributed by
Mayborn (UK) Limited,
Balliol Business Park,
Newcastle upon Tyne,
NE12 8EW.

EU Representative,
Mayborn France
Sarl 56 Rue de Paris,
Boulogne Billancourt,
92100, France.

All IP rights held by Mayborn (UK)
Limited and/or its affiliates.

TOMMEE TIPPEE®, FLEXIPEN™,
Copyright © 2022.

42304060 12250722_3

Any questions?
Visit our website: tommeetippee.com

Please read and keep for future reference
Veuillez lire et conserver ces instructions
Léa y conserve estas instrucciones

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EN |

WARNINGS AND PRECAUTIONS

- This thermometer is not intended as a substitute for consultation with your doctor.
- Keep out of reach of children aged under 12 years.
- Do not allow children to take their own temperature unattended.
- To avoid risk of choking from swallowing small parts or batteries please keep the device out of the reach of children and pets.
- No modification of this equipment is permitted.
- No parts should be removed except when replacing a battery.
- Intended for household use only.
- Keep away from direct sunlight and heat sources as well as strong electromagnetic sources to avoid interference.
- Normal body temperature is a range which varies by the area of the body at which the temperature is taken. Therefore readings from different sites should not be directly compared. The range of normal can also vary from person to person so it is important that you determine your normal temperature range. We recommend that you practice taking temperatures on yourself and healthy family to pre-determine normal temperature ranges.

FUNCTIONS

In-Mouth

The mouth should remain closed for up to 5 minutes before taking a reading. Place the probe well under the tongue. Instruct child not to bite the probe. Measurement time = 8-15 seconds.

Under Armpit

The armpit is not an ideal site to measure body temperature because you're measuring the surface of the skin and not a closed body cavity. Ensure the armpit is dry. Place the thermometer under the armpit and press arm tightly next to the body. Ensure the sensor is covered under the armpit. We recommend a longer measuring time of 60+ seconds for this method.

HOW TO USE

- Please make sure the environment temperature is between 16°C - 31°C before use.
- Press power switch to turn the unit on.
- It will beep when you turn it on to indicate sufficient power and you will see this full segment screen for a few seconds:

- After the full segment screen, you will see the last measured value with "Last Memory" on the display.
- After a few more seconds the screen will display **Lo°C** or **Lo°F**. Then the °C or °F symbol will flash. Please wait for the symbol to flash 3 times before taking a measurement.
- The thermometer will beep when its finished taking a reading:

<i>Bibi - bibi - bibi</i> with green back light	normal reading	Below 37.6°C / 99.7°F
<i>Bibibibi - bibibibi - bibibibi</i> with red back light	fever alarm	Above 37.6°C / 99.7°F

- During measurement LCD will display **Lo°C** or **Lo°F** if the measured temperature is below 32°C / 90°F or will display **Hi°C** or **Hi°F** if the measured temperature is above 43.9°C / 111°F
- Turn off the unit by lightly pressing the power switch, or leave it for a few minutes and it will auto-off to save power.
- To maintain the accuracy of each measurement, please wait for at least 3 minutes before taking another measurement.

How to toggle between °C and °F:

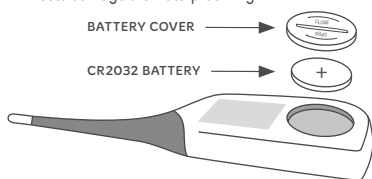
Press switch to turn on and hold it down until the display shows Lo°C (with no flash), release the switch and within 2 seconds press it again. You do this to change from °C to °F or back.

CARE AND CLEANING

- Make sure the sensor is clean to ensure an accurate reading.
- We recommend you clean the sensor before every measurement.
- The thermometer is waterproof and can be wiped clean with water or cleaning lotion.
- We recommend using a 70% alcohol swab or cotton wool moistened with 70% alcohol to gently wipe the probe and sensor clean.
- Do not immerse in boiling water for sterilising.

REPLACING THE BATTERIES

- If the display becomes blurred or if this symbol appears on the LCD , the battery must be replaced.
- To replace the battery, open the cover using a coin and remove the battery.
- Replace with a new battery (CR2032) then re-insert the cover with the coin, taking care not to damage the waterproof ring.

- To prevent battery corrosion, remove batteries from unit if you don't intend to use it for some time.
- Batteries should always be removed or replaced by an adult. Keep batteries away from children. Remove batteries if the Thermometer will not be used for some time.

PRODUCT SPECIFICATION

Measuring Range: 32.0°C~43.9°C (90°F~111°F)
 Measuring time: About 8~10 seconds
 Accuracy: ±0.1°C (±0.2°F) for 35.0~42.0°C (95°F~107.6°F) & ±0.2°C (±0.4°F) for remaining range at an ambient of 22°C (71.6°F)
 Display: 31/2 LCD digits (Unit of display: 0.1°C/0.1°F)
 Battery: CR2032 lithium
 Battery life: About 1000 times
 Weight: 27 g

SYMBOL DESCRIPTIONS

The CE mark and Notified Body Registration Numbers, the requirement of Annex II from Medical Device Directive 93/42/EEC are met.

Indicates this device is subject to the Waste Electrical and Electronic Equipment Directive in the European Union. To protect the environment, dispose of useless device at appropriate collection sites according to national or local regulations.

Battery recycling.

Please read the instructions for use.

Authorized representative in the European community.

Wellkang Limited
 Enterprise Hub, NW Business Complex, 1 Beraghmore Road, Derry, BT48 8SE, Northern Ireland, UK
 16 Castle St, Dover, CT16 1PW, England, UK

Cotronic Technology Limited,
 Floor 4-6, Block 7, West of ZhouShi Road, XiXiang Street, Bao An Zone, ShenZhen, GuangDong, China.
 Tel: +852 2602 2021

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