Instructions for use

made for me[®]

in-bra wearable breast pump

Gebrauchsanweisung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso
Instruções de utilização
تعليمات الاستخدام
使用说明书

0423642/3
Model: 1203
Version No: C-1

Hands down, hands-free is best

With all the performance you'd expect from a regular electric breast pump, our wearable design fits comfortably in your bra so you can go about your day with total freedom.

Questions?

For further information or support, please scan below to visit our consumer support page at tommeetippee.com

Or join the conversation

Please download the Tommee Tippee app which is available on Android and iOS app store.

EN

Contents

1. Indications for use	2
2. Contraindications	2
3. Warnings	2
4. Precautions	3
5. Device description	5
6. Operating instructions	7
7. Cleaning instructions	14
8. Troubleshooting	16
9. Technical information	19
10. Radio Frequency Wireless Technology in Medical Devices	19
11. Storage conditions	20
12. Use-life	21
13. Dispose of the device	21
Appendix: EMC Information	21
DE 1. Schnellanleitung	28
FR 1. Guide de connexion rapide	34
ES 1. Guía de conexión rápida	40
IT 1. Guida al collegamento rapido	46
PT 1. Guia de ligação rápida	52
CN 1. 快速连接指南	58
65	دليل التوصيل السريع .1 AR

Tips:

- a) *A warning describes serious adverse reactions and potential safety hazards, limitations in use imposed by them, and steps that should be taken if they occur.*
- b) *A precaution includes information regarding any special care to be exercised by the practitioner and/or patient for the safe and effective use of the device.*
- c) *A contraindication describes conditions in which the device should not be used because the risk of use clearly outweighs any possible benefit.*

1. Indications for use

The Wearable Breast Pump is a powered breast pump to be used by lactating women to express and collect milk from their breasts. It is intended for a single user.

2. Contraindications

None known.

3. Warnings

1. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
2. Powered breast pumps that are designed for single users should never be rented or shared.
3. Pregnant women should not use the breast pump, as pumping can induce contractions or premature labor.
4. Do not use breast pump while sleeping or overly drowsy.
5. Do not modify the device without authorization of the manufacturer. This could cause improper functioning.

6. Do not replace the battery by yourself, otherwise incorrect replacement of the battery will result in battery damage, even fire, explosion. If necessary, please feel free to contact our customer service staff and it will be repaired or replaced by our qualified personnel.

7. Please charge the batteries if you have not used the breast pump for more than 2 weeks.

8. This device is not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

4. Precautions

1. To protect against fire, electric shock and injury to persons, do not immerse motor unit, USB cable or air tube in water or other liquid.

2. Do not place the air tube in microwave or electric steam sterilize or boiling water as it will melt.

3. Before each use visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.

4. If the power adapter casing or wiring becomes loose, separated, or frayed, stop use of the power adapter immediately and contact the device manufacturer. Stop using the device immediately if you note any smoke or burning from the pump unit or power adapter.

5. Tubing - Inspect the tubing. If you find condensation in the air tube, wash and dry it immediately or replace the tubing. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or disinfect the tubing or tubing connection. Instead contact the manufacturer.

6. Misuse of the breast pump may result in the risk of fire, electric shock and/or personal injury.


7. This device should not be used or played with by children. Keep the pump, its air tube and USB cable out of reach of children.

8. Do not leave the device in direct sunlight.

9. Do not allow product to come into contact with oil-based foods (e.g. oil/ tomato based sauces) as staining may occur.
10. Before use, ensure that your hands and the breast pump are clean.
11. Do not turn the teat inside out for cleaning. Damage could result. Use a teat brush.
12. For safety and hygiene reasons it is recommended that the feeding teat is replaced at least every 2 months.
13. Do not use any accessories other than those supplied with this appliance.
14. Only use the adapter and USB cable supplied with this product.
15. Do not use while in transit.

5. Device description

The Tommee Tippee Wearable Breast Pump comes with everything you need to prepare, pump, feed and store milk.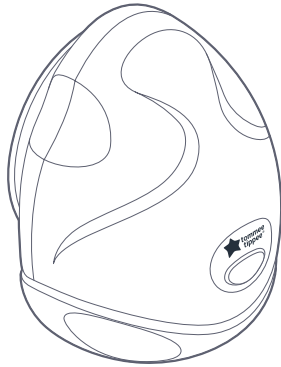


Parts of Tommee Tippee Wearable Breast Pump

Part	Single	Double
A	1 x 24mm horn / flange	2 x 24mm horns / flanges
B	1 x 20mm horn / flange	2 x 20mm horns / flanges
C	2 x bra adjusters	2 x bra adjusters
D	1 x slow flow nipple / teat	2 x slow flow nipples / teats
E	1 x screw ring	2 x screw rings
F	1 x pump body	2 x pump bodies
G	2 x upper lids	4 x upper lids
H	2 x diaphragm assemblies	4 x diaphragm assemblies
I	2 x lower lids	4 x lower lids
J	2 x duck valves	4 x duck valves
K	2 x collector bottles	4 x collector bottles
L	1 x USB charging cable and multi-region plug	1 x USB charging cable with 2 heads (shown opposite) and multi-region plug
M	1 x carry bag	1 x carry bag
N	nipple size guide	nipple size guide
O	1 x 17mm horn / flange insert	2 x 17mm horn / flange inserts
P	1 x 19mm horn / flange insert	2 x 19mm horn / flange inserts

Tip: Before using your pump for the first time, we recommend a full charge using the cable and plug provided. To ensure you get the most comfortable fit and optimized milk let down, we also recommend using the nipple fit guide included to find which horn or insert size works for you.


b. Parts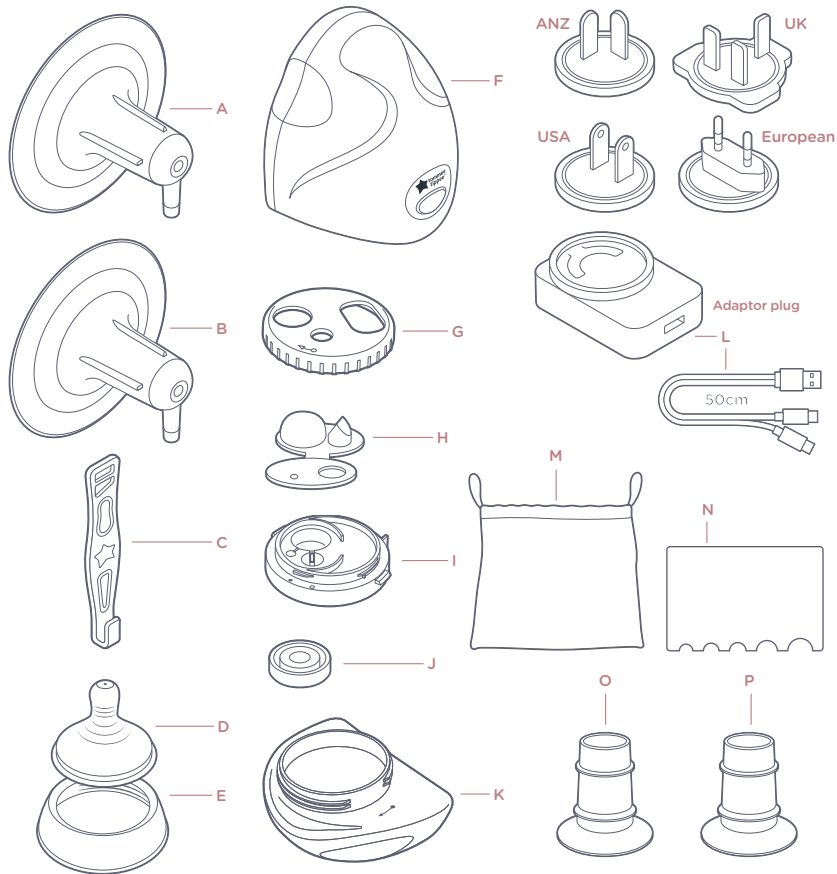

6. Operating instructions

a. On-pump controls

1. Turn on
light showing white (offline) or blue (paired)
2. Pairing
press and hold power button to enter pairing mode (flashing blue light)
3. Choose breast
light showing on L or R side to denote breast
4. Press start/stop
light showing white
5. Battery
each state (solid green, red, pulsing red)
6. Battery (charging)
(solid green or red pulsing)
7. Intensity
left or right toggle - no change to illumination

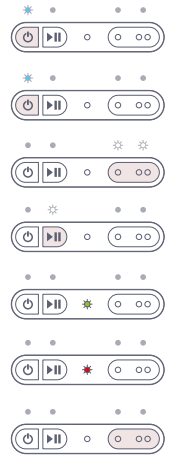b. Charging

Charge your pump using the cable and adaptor plug (L) provided before first use. The USB port is protected by a dust cover and hidden beneath the horn. Disassemble the pump to charge. For more information, read the Charging section on the following pages.

c. Horn / flange sizing


We include 24mm and 20mm horns / flanges (A/B) and 17mm and 19mm horn / flange inserts (O/P) so you can choose the one that is most comfortable for you. With the horn / flange disassembled from the pump body, place it over your breast to assess fit. We recommend measuring the diameter of your nipple using the Tommee Tippee Nipple Size Guide (N) included in the pack so you're familiar with your size. You may find an insert is required for best results. If you need other sizes, please visit the Tommee Tippee website where you can order free of charge alternatives.

d. Turning it on

Once fully assembled, press the power button  to turn the pump on. Check the 'Quick Connection Guide' for instructions on how to pair your pump to the Tommee Tippee app.

e. Aligning your nipple

Correct nipple alignment is a really key factor in milk production, much like a deep latch in breast feeding. Your new pump includes a unique nipple alignment guiding light that makes it easy to align your nipple, even in low-light. Turn your pump on and direct the red spotlight towards your nipple. Hold your breast with your other hand and move the pump closer to your nipple, ensuring the centre of the spotlight stays in the centre of your nipple until the pump is in place.

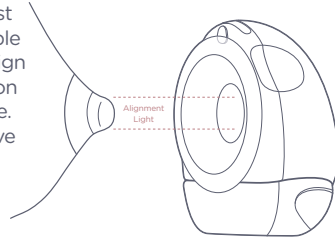f. Starting your session

Once the pump is turned on and in position, press play/pause to start pumping. If you need to pause pumping at any point, press play/pause. For a more discreet and convenient user experience, use your Tommee Tippee app to play/pause.

g. Choosing your intensity and mode

Your wearable breast pump offers two modes – massage and expression. We know every mum and every day is different so, in the app, you can move between modes with the tap of a button at any point during your session. Please note, it can take time for the pressure to drop when going down levels.

Massage mode (also known as stimulation mode) uses a higher frequency, lower intensity suction which mimics how your baby first stimulates the breast to encourage milk flow.

Expression mode uses slower sucks to draw your milk out and into the collector bottles effectively. Some prefer to alternate, others find one setting and stick with it. In expression mode, there are 8 intensity settings. Choose a level that is comfortable for you. Remember this may well change per session and even per breast. Remember a higher intensity setting does not always mean a higher volume of milk. Your comfort is more important.

In the app, you can switch between massage and expression mode whenever you like because every person and every session is different.

h. Automatic functions

Your wearable breast pump will automatically pause when the bottle is full. If you want to continue to pump, you will need to submit the session so it is registered in the analysis page in your app before you empty the collector and start a new session. As a safety feature, we've also limited session duration to 30 minutes, at the end of which the pump will pause and you will be unable to restart. If you do want to continue, you will need to start a new session. The pump will also pause if you lean too far over in order to reduce the risk of leaking. This is unlikely to occur in general use but if it does, simply sit/stand back up straight and resume pumping. The lights on the pump (nipple alignment spotlight and button lights) will automatically turn off after 1 minute during use for discretion. Press any button to re-illuminate the lights.


i. Ending your session

Once you have finished your session, pause and finish the cycle and follow instructions in the app to collect every last drop, leaning forward for a moment before using your finger to break the seal between pump and breast. At this stage you can also manually alter the volume if you find the amount shown in the app doesn't quite match up to what's in the collector bottle. Foremilk has a lighter consistency than hindmilk which can impact the accuracy of the reading. Once the session has been added to the analysis page, you can simply turn the pump off, wash and disinfect the parts that require it and charge the pump body as required. When you're ready to remove the collector bottle, simply press the large button at the front of the pump to release the latch that holds the bottle in place.

j. Direct-to-feed and storage


To make it even easier to feed breast milk once you've expressed, the milk collector is compatible with our nipples / teats so you can use like a bottle. Simply assemble the nipple / teat to the screw ring, attach to the milk collector and you're ready to feed.

To store, simply remove the collector bottle sub-assembly from the pump and put in the fridge or freezer as preferred. If freezing, ensure the bottle isn't over-filled and does not exceed the maximum capacity (5 fl oz). Alternatively, store your breastmilk in line with healthcare guidelines.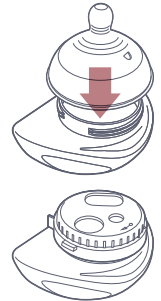


Assembly

Your pump arrives assembled. Please disassemble, clean and disinfect before first use. See next section for cleaning and disinfection instructions.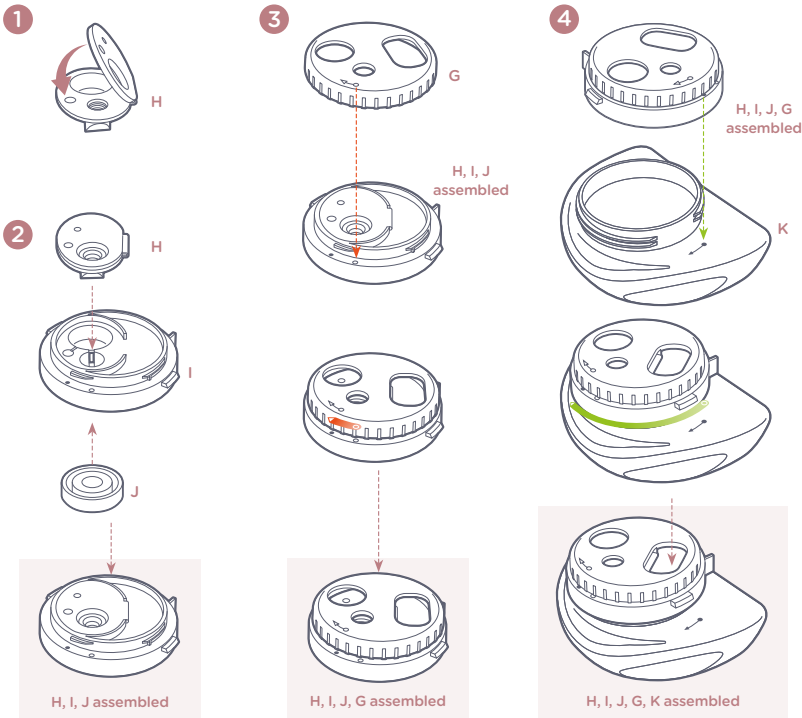

To assemble, choose your preferred horn / flange size and insert into the pump body (1). If you need a different size, please contact the Tommee Tippee customer support team. With the collector bottle components assembled, clip into the bottom of the pump body (2).

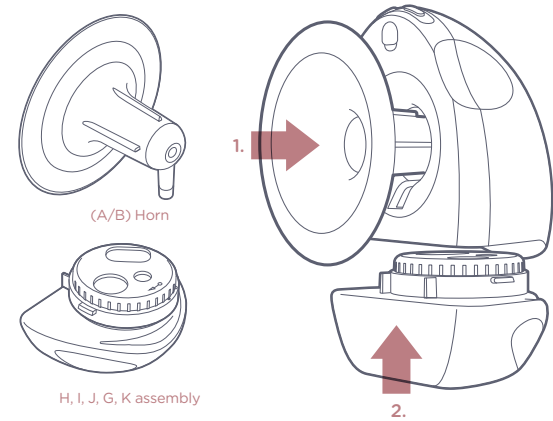Charging

It's important to fully charge your pump when the battery is low (indicated by a red light).

To charge the pump, remove the collector bottle and horn/flange. On the back of the pump body, pull and twist the small USB port cover to expose the port. Plug the USB-C cable into the port and plug the other end into a wall socket using the adaptor plugs provided.

When charging the pump, wait for the battery light to turn green. This indicates that the pump is fully charged and ready to use. The pump cannot be used whilst charging. You should only use the cable and plug provided with this product. A full charge offers up to 4 hours of use per pump. Please make sure the device is turned off before plugging in the USB cable to charge. The device turns off automatically after 10 minutes of idle time.

Quick connection guide

If you can't wait to get started with your new pump, here's a handy guide to help you get the most out of it.

Please download the Tommee Tippee app which is available on android and iOS app store.

iOS

Use QR code to download app for iOS device

Android

Use QR code to download app for Android device

For more information, tips and tricks on using the app, visit tommeepee.com and search 'in-bra wearable'.

Your pump can also be used offline. Simply press the power button and press either the left or the right button to adjust intensity. In offline mode, the power button shows solid white.


Pairing

Once you've downloaded the Tommee Tippee app and registered your account, pairing is really simple.

You will need to pair each pump separately.

Follow the below instructions to pair the device.

1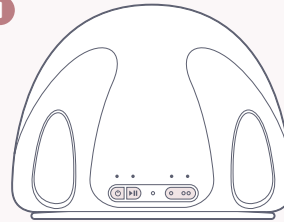

Press and hold the power button  on the top of the pump for 2 seconds

Tip: Before using your pump for the first time, we recommend a full charge using the cable and plug provided.

2

The pump should appear as a pop up on your app home screen, allowing you to pair with the tap of a button.

If no pop up appears, you just need to press the  in the top right corner and select the pump. Your phone will search for the pump and pair it. Once paired, the pump is ready to use. We recommend that you keep your phone close by with you so that it doesn't go out of range.

7. Cleaning instructions


Your wearable breast pump must be cleaned and sterilized before first use. The pump body (F) should not be washed or sterilized. Always disassemble components before cleaning or sterilization.

All of the parts intended for washing can go in the dishwasher and can be disinfected, including with UV. The pump body (F) should not be washed or disinfected. It contains electronics that must be kept dry. You can use a clean, damp cloth to wipe it as required.

Components that can be washed and sterilised:

Part	Single	Double
A	1 x 24mm horn / flange	2 x 24mm horns / flanges
B	1 x 20mm horn / flange	2 x 20mm horns / flanges
C	2 x bra adjusters	2 x bra adjusters
D	1 x nipple / teat	2 x nipples / teats
E	1 x screw ring	2 x screw rings
G	2 x upper lids	4 x upper lids
H	2 x diaphragm assemblies	4 x diaphragm assemblies
I	2 x lower lids	4 x lower lids
J	2 x duck valves	4 x duck valves
K	2 x collector bottles	4 x collector bottles
O	1 x 17mm horn / flange insert	2 x 17mm horn / flange inserts
P	1 x 19mm horn / flange insert	2 x 19mm horn / flange inserts

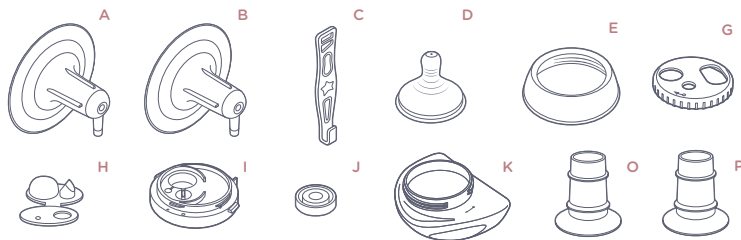Cleaning

For dishwasher

After every use, follow these directions to clean with dishwasher:

1. **Take apart.** Separate all bottle parts (especially parts G, H, I and J)
2. **Rinse.** Rinse bottle parts and any other feeding items by holding them under running warm water.
3. **Wash.** Place bottle parts and other feeding items in the dishwasher. **If possible, run the dishwasher using hot water and a heated drying cycle (or sanitizing setting); this can help kill more germs.**
4. **Remove from dishwasher.** Wash your hands with soap and water before removing and storing cleaned items.

If items are not completely dry, place them on a clean, unused dish towel or paper towel to air-dry thoroughly before storing in an area free of dust or dirt. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.

For cleaning by hand

After every use, follow these directions to clean by hand:

1. Rinse in cool water all separated parts that came in contact with breast and breast milk in order to remove breast milk residue.
2. Soak all separated parts in warm soapy water for 5 minutes.
3. Clean with a clean dish-cloth or soft brush.
4. Rinse all separated parts with clear water.
5. Allow all parts to air dry in a clean area.
6. Store dry parts when not in use.

Sterilize/sanitize

1. Fill a pot with enough water to cover all parts.
2. Bring water to a boil.
3. Place parts in boiling water for 10 minutes.
4. Allow water to cool and gently remove parts from water with tongs.
5. Allow all parts to air dry in a clean area.
6. Store dry parts when not in use. Do NOT store wet or damp parts.

Other forms of sterilization/sanitization may also be used e.g. microwave, steam or UV.

Electric Parts Cleaning

If you need to clean the power unit or USB cable, wipe the outside with a clean damp cloth.

8. Troubleshooting

How do I find my fit?


Finding the right fit is incredibly important because it will help you express more milk in less time and will result in a more comfortable experience. The “right” flange size can vary depending on lots of factors like your breast tissue, skin elasticity, nipple shape, nipple damage and personal preference, but as a general rule, experts agree you should add 3 to 4 mm to your nipple diameter, not including your areola, to determine size. Our Nipple Size Guide takes the guesswork out of it so use that to determine size.

The sizes in the pack don't fit, what now?

We offer a 28mm horn / flange free of charge. Scan the code on top of your pack or contact Tommee Tippee to order your size.

How can I make it more comfortable in my bra?

We'd always recommend having your breasts measured when you have a baby. You'll be amazed how different your new breasts can be. The pump should fit comfortably in a sturdy nursing bra but will be less secure in a low cut or non-nursing style. The pump should compress your breast tissue enough to generate a seal but should be comfortable. If it isn't, we include bra adjusters in the pack (C) which can be clipped onto your nursing bra and have the effect of extending your cup.

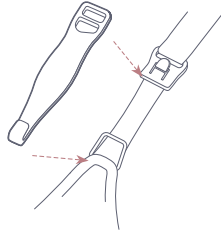What settings should I use?

The ConstantComfort™ technology in this pump is different to every other pump you've tried. Because of that we recommend spending some time finding the setting that works best for you. Choose between massage and expression mode and expression intensity. Start on level 1 and increase every 30-40 seconds until you find a setting that matches comfort with output. If you feel it getting too intense, drop back down. Remember every day and every session is different so don't be afraid to change your setting to suit your needs.

What is ConstantComfort™?

ConstantComfort is the name we give to our patent-pending technology that more accurately mimics a breast feeding baby. It combines a low level of constant vacuum around the nipple with small waves of pressure to draw out milk. This is the equivalent of baby latching on with their mouth (vacuum) and using their tongue to draw milk from the nipple (pressure waves). Not only does this mean a gentler pressure profile compared to other pumps in the market that use a “sawtooth” pattern, it also means the pump is held more securely in place so there's less risk of the pump slipping and the nipple mis-aligning. Given that this is a hands-free wearable, that's a massive benefit as it allows you to go about your day without having to constantly re-adjust.

Why is the amount the app tells me I've pumped different from how much is in the collector bottle?

The pump uses a sensor to estimate the amount of milk in the collector bottle. This can be affected by a number of things like milk consistency (foremilk vs. hindmilk), movement, angle of use and sensor interference. Ensure the small black sensor is free of debris and sit/stand upright and stationary at the end of your session for best results.

How do I convert the collector bottle into a feeding bottle?

Simply remove the diaphragm and lid assembly from the top of the collector bottle, assemble the nipple / teat and screw ring and screw in place.

How do I store milk in the collector bottle?

Your pack comes with spare parts so the entire collector bottle assembly can be stored in the fridge or freezer until you are ready to feed with it.

Why won't my pump pair to the app?

The pump uses Bluetooth to pair to your phone. Download the Tommee Tippee app, register an account and log in. Turn on the pump and then press and hold the power button until the LED light flashes blue. You will see a pop-up appear in the Tommee Tippee app and can then add the pump. Please note you will need to do this for each pump if you have bought a pair.

What if I'm not getting enough milk?

There are lots of reasons why your milk production fluctuates over time but if you know you're getting less than “normal” there are a few things to try.

- Check the alignment of your nipple in the pump – it's important that the centre

of your nipple is inserted at the centre of the horn / flange.

- Check the size of the horn / flange you are using – it's important that the horn / flange feels comfortable and doesn't draw in too much of your breast tissue.
- Check the seal when the pump is running – the pump should feel very secure once the program starts.
- Check the fit in your bra – if the pump feels “squashed” against your breasts, try one of the provided bra adjusters to loosen the fit slightly.
- Check the intensity settings – finding a setting that is comfortable and effective will produce the best combination of comfort and output.

I can't get the last few droplets of milk out of the collector bottle?

We recommend using the teat/nipple that comes with the pump to feed direct from the collector bottle as this is the most effective way of removing milk. If decanting for storage or feeding from another bottle, you may find some milk remains after pouring. Some users have found extracting with an infant dropper an effective way of collecting the last few droplets of milk.

Why is my Bluetooth connection “lost”?

If you close the app during a session you may find when you go back in that the pump appears “offline”, even though it's still expressing milk. Use your thumb to “drag down” on the screen to refresh and you should see “no Bluetooth” icon disappear. If you find this happens a lot and refreshing does not help, please contact Tommee Tippee.

How do I upgrade my firmware?

The Tommee Tippee In-Bra pump is smart which means it can evolve over time, even after you've purchased it. Where it results in a better experience for the consumer, we will update the app periodically. It's important that you keep the app up to date so you have access to the latest features. In the app, you will receive a notification if there is an update available so we recommend keeping the app downloaded on your phone.

Why doesn't the suction feel like it did?

There are two valves in the milk collector bottle assembly. They can be a little sticky if not properly cleaned so we recommend pinching them to ensure they can open prior to assembly. The other thing to ensure is that the collector bottle is inserted firmly in the bottom of the pump. Finally, try charging your pump fully to see if that resolves the problem.

How do know if the diaphragm assembly is OK?

Check the duck-bill valves are clean and open easily. Give them a gentle squeeze if they're stuck.

9. Technical information

- Model: 1203
- Input: 5V \approx 1A
- Battery Type: Built-in lithium-ion battery
- Voltage: 3.7 V \approx
- Battery Capacity: 1400 mAh
- Device Frequency Band: 2500-2600 kHz
- Device Max Output RF Power: 10.5 dBm
- Adapter Information;

Input: 100-240V-50/60Hz 0.4A

Output: 5.0V \approx 1.0/ 2.0A 10.0W

10. Radio Frequency Wireless Technology in Medical Devices

Bluetooth is applied only for fulfilling the wireless control on the breast pump. The technology complies with Bluetooth Low Energy V5.0 for 2402-2480MHz band. The characteristic of the modulation is GFSK, and the effective radiated RF power is 7dBm, the maximum transmit power is 10.5dbm. According to Bluetooth Low Energy V5.0 standard, the data throughput is 0.305Mbps, latency is about 100ms, and the data integrity is -85dBm@10% PER.

The QoS risk of the Bluetooth is acceptable for the device, since the operating parameters can be set on the device, and the device will follow the last directive operated on the device. Access to the breast pump is accepted only after necessary pairing between the breast pump and the smart device is completed. Possible unauthorized monitoring over the Bluetooth communication might happen to modify the device operating parameters and steal device operating

data. Manipulation on the operating parameters might cause the device to deliver excessive or inadequate vacuum, consequently lead to injury or non-effects to the user.

We have taken the following control measures:

1. The software is designed to allow authorization to manipulate the device operating parameters only after Bluetooth connection is accepted by pressing certain button on the device;
2. The software is designed to allow authorization to acquire the device operating data only after Bluetooth connection is accepted by pressing certain button on the device.

If other devices (such as computer, mobile phone with Bluetooth function) want to establish a wireless connection with the system, the premise is that they must carry out Bluetooth pairing via pressing certain button on the device.

Please take attention that changes or modification not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Federal Communications Commission (FCC) Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

11. Storage conditions

Store the Wearable breast pump in place where no direct sunlight, near a source of heat or contact with solvents or harsh chemicals.

12. Use-life

Pump is reusable for a single user, and has an expected operating life of 12 months.

13. Dispose of the device

The breast pump contains a rechargeable lithium ion battery. Use your local return and collection system for electrical waste and electronic equipment (including batteries).

MAYBORN (UK) LIMITED, MAYBORN HOUSE, BALLIOL BUSINESS PARK, NEWCASTLE UPON TYNE, NE12 8EW, UNITED KINGDOM.

Appendix: EMC information

Guidance and Manufacturer's Declaration

Below cables information are provided for EMC reference.

Cable	Max. cable length, Shielded/unshielded		Number	Cable classification
DC Power Line (USB cable)	0.5m	Unshielded	Set 1	DC Power

Important information regarding Electro Magnetic Compatibility (EMC)

This electrical medical equipment needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual; The equipment conforms to this IEC 60601-1-2:2014 standard for both immunity and emissions. Nevertheless, special precautions need to be observed:

- The equipment with Following ESSENTIAL PERFORMANCE is intended used in Home healthcare environment.

ESSENTIAL PERFORMANCE: the breast pump with anti backflow design, max suction pressure is not exceed -300mmHg.

· WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

·The use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

· WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of this product, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

· WARNING: If the use location is near (e.g. less than 1.5 km from) AM, FM or TV broadcast antennas, before using this equipment, it should be observed to verify that it is operating normally to assure that the equipment remains safe with regard to electromagnetic disturbances throughout the expected service life.

· STATEMENT: For the purpose of its operation, the equipment has wireless communication function, it includes RF transmitter and receiver, 2.4GHz, Pulse modulation.

EMI Compliance Table (Table 1)

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	Home healthcare environment
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	Home healthcare environment

EMI Compliance Table (Table 2-4)

Table 2 - Enclosure Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3
	IEC 61000-4-8	30A/m 50Hz or 60Hz

Table 3 - Proximity fields from RF wireless Communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Home healthcare environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ±5kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810	800-960	Pulse modulation 18Hz, 28V/m
870		
930		
1720	1700-1990	Pulse modulation 217Hz, 28V/m
1845		
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240	5100-5800	Pulse modulation 217Hz, 9V/m
5240		
5785		

Table 4 - Input a.c. power Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrical fast transients/burst	IEC 61000-4-4	±2 kV 100kHz repetition frequency
Surges Line-to-line	IEC 61000-4-5	±0.5 kV, ±1 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands and amateur radio bands between 0.15MHz and 80MHz 80%AM at 1kHz
		0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
Voltage dips	IEC 61000-4-11	0% UT; 1 cycle and 70% UT; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% UT; 250/300 cycles

Safety information

NATURAL START® SLOW FLOW NIPPLE / TEAT AND RING

Please read and retain the instruction guide for future reference as it includes important information.

For your child's safety and health

WARNING!

- Always use this product with adult supervision. Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or if the product has disassembled.
- Never use feeding nipples / teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle for long periods through the day and particularly through the night when saliva flow is reduced or if it is used as a soother.
- Always check food temperature before feeding. Heating in a microwave oven may produce localised high temperatures. Take extra care when microwave heating. Always stir heated food to ensure even heat distribution and test the temperature before serving.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- The hygiene cap can pose a suffocation hazard-keep out of the reach of children.

CLEANING & STERILISATION

- Before first use place in boiling water for 5 minutes. This is to ensure hygiene.
- Clean before each use.
- Always wash the product immediately after use.
- Wash in warm soapy water and rinse in clean water or use a dishwasher (top shelf only).
- Do not clean with solvents or harsh chemicals.
- Do not allow product to come into contact with oil-based foods (e.g. oil/tomato based sauces) as staining may occur.
- Do not turn the nipple/teat inside out for cleaning. Damage could result.
- Use a nipple / teat brush.

STORAGE AND USE

- Inspect before each use and pull the feeding nipple/teat in all directions.
- Do not store or leave in direct sunlight or near a source of heat.
- Do not store with or allow to come into contact with solvents or harsh chemicals.
- Not suitable for use in a conventional oven or a combination microwave oven with the grill feature switched on.
- Do not leave a feeding nipple/teat in direct sunlight or heat or leave in disinfectant ("sterilising solution") for longer than recommended as this may weaken the nipple / teat.

- For safety and hygiene reasons it is recommended that the feeding nipple/teat is replaced at least every 2 months.
- Do not heat in a microwave with bottle sealed. Remove nipple / teat, locking ring and dome cap.

MADE FOR ME® WEARABLE BREAST PUMP

IMPORTANT WARNINGS!/IMPORTANT SAFEGUARDS

Please keep this instruction guide for future reference as it includes important information.

You can also find these instructions at tommeetippee.com.

- To protect against fire, electric shock and injury to persons, do not immerse motor unit, USB cable and adapter, or air tubes in water or other liquid.
- Do not place the power unit in a microwave oven or steam steriliser.
- DO NOT WASH the air tubes or adapter. This will result in bacteria build up and can lead to pump failure. Washing or sterilising the air tubes will invalidate the warranty and will damage the power unit.
- Do not place the air tubes in a microwave or electric steam steriliser or boiling water as they will melt.
- Misuse of the appliance may result in the risk of fire, electric shock and/or personal injury.
- Do not clean with solvents or harsh chemicals.
- This appliance should not be used or played with by children. Keep the pump, its air tubes and USB cable out of reach of children under 8 years old.
- The appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- User maintenance, other than cleaning and sterilising is not required.
- Do not open the power unit, it contains no user serviceable parts and will invalidate the warranty.
- Do not operate any appliance with a damaged USB cable or motor casing. If the appliance malfunctions or has been damaged please contact tommeetippee.com.
- If the damage is from new the unit must be returned to manufacturer.
- Do not leave the appliance in direct sunlight.
- Do not store below freezing or above 45°C.
- If damaged within a retail warranty period, this appliance should be exchanged at place of purchase. The manufacturer will then be made aware.
- Do not use any accessories other than those supplied with this appliance.
- The appliance should not be used adjacent or stacked with other equipment as the two appliances may interfere with each other.
- Do not allow appliance to come into contact with oil-based foods (e.g. oil/ tomato based sauces) as staining will occur.
- This appliance is a personal care product and therefore cannot be returned once opened, unless faulty.

- Only use the adapter and USB cable supplied with this product.
- Do not use whilst in transit.

For domestic use only - not intended for commercial or institutional use.

SAVE THESE INSTRUCTIONS

Any questions? Visit our website tommeetippee.com.

Technical Information

- Model: 1203
- Input: 5V = 1A
- Battery Type: Built-in lithium-ion battery
- Voltage: 3.7 V =
- Battery Capacity: 1400 mAh
- Device Frequency Band: 2500-2600 kHz
- Device Max Output RF Power: 10.5 dBm
- Adapter Information:
Input: 100-240V-50/60Hz 0.4A
Output: 5.0V = 1.0/ 2.0A 10.0W

Working Condition

Rated temperature range: 5°C to 40°C (41°F to 104°F)
Rated relative humidity: 20%-80%
Rated atmospheric Pressure: 850Hpa - 1060Hpa
Long-term storage should be kept in a dry and cool place

Transport & Storage condition

Temperature: -20°C to 55°C (-4°F to 131°F)
Humidity: 20% - 80%
Atmospheric pressure: 850Hpa - 1060Hpa

Regulatory Information

Information on the Disposal of Waste Electrical and Electronic Equipment (WEEE).

This symbol means that your product shall be disposed of separately from household waste in accordance with local laws and regulations. When this product is ready for disposal, take it to a collection point designated by local authorities. Separating and recycling of your product at the time of disposal will help conserve natural resources and ensure that the product is recycled in a manner that protects human health and the environment. Penalties may be applicable for the incorrect disposal of this product in accordance with national legislation.

This product contains a battery which is permanently built into the product for safety and performance reasons. The battery should not require replacement during the lifetime of the product. It should only be removed by skilled service personnel. Dispose of this product as electrical waste to ensure the correct waste treatment of the battery.

RED Declaration

Mayborn (UK) Limited hereby declares that this product is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU and all other applicable EU directive requirements. The complete declaration of conformity can be found at: www.tommeetippee.com/en-gb/product-support

Made in China

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 off the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference with radio communications in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. This device must accept any interference received, including interference that may cause undesired operation.

If this equipment does cause harmful interference to radio, television, or other devices, which can be determined by turning the requirement off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the other devices(s) are connected
- Consult the dealer, manufacturer or an experienced radio / TV technician for help.

Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment

IP22

IP22 Indicates that the unit is resistant to condensation and finger entry.

Compliance with Federal Communications Commission Requirements.

Keep the device dry.

The product contains a Lithium Ion Battery.

Voltage 5V= Power 10W

The power and voltage consumed by the device when plugged into a USB socket.

The product is double insulated (Class II appliance).

In some countries this is a medical appliance and this indicates that this product has Applied Parts.

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Mayborn (UK) Ltd
Mayborn House, Balliol Business Park, Newcastle Upon Tyne NE12 8EW
UNITED KINGDOM

Mayborn USA Inc.
1010 Washington Boulevard, 9th Floor, Stamford, CT 06901
UNITED STATES OF AMERICA

EU Representative
Mayborn France Sarl, 56 Rue de Paris, Boulogne Billancourt, 92100
FRANCE

Mayborn ANZ Pty Ltd.
(ABN 66 154 703 134) PO Box 475, Mt Waverley VIC 3149
AUSTRALIA

Let us know what you think

We'd love to hear what you think of our
wearable breast pump. Share your photos,
reviews and tell us about your
experiences on our social channels.

Any questions?
Visit our website: tommeetippee.com

TOMMEE TIPPEE® MADE FOR ME® Copyright © 2024.
All IP rights held by Mayborn (UK) Limited.